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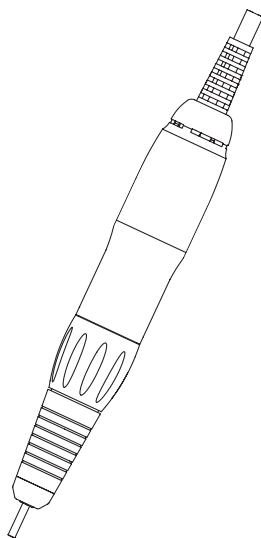
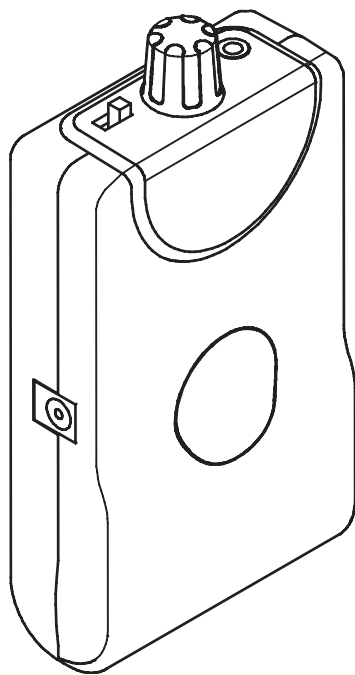
Product Instructions

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In order to prevent the physical injury and the damage of this product, please make sure to read the operation manual and product safety information of this instruction before using. Please keep this instruction at the place where all the users can read any time to become well-informed.

SIMON KELLER LTD. is not responsible for the damage and loss caused by following reasons:

- | In case user did not follow in accordance with this instruction
- | In case of external factors such as inappropriate installation
- | In case of any other inappropriate usage
- | In case of repair by uncertified person

Contents

- 1. Product Composition 18
 - 1.1 Product Composition

- 2. Examples of usage 19
 - 2.1 For Portable
 - 2.2 For Desk

- 3. Operation 20
 - 3.1 Part Function
 - 3.2 LED LAMP detail
 - 3.3 Overload Usage Preventing Function

- 4. Battery 21
 - 4.1 Battery Recharge
 - 4.2 Battery Exchange

- 5. Maintenance 23
 - 5.1 Hand Piece Burr Exchange
 - 5.2 Ball Bearing Exchange
 - 5.3 Carbon-Brush Exchange
 - 5.4 Collet Chuck Exchange
 - 5.5 Handpiece Diagram

- 6. Product Specifications 26

- 7. Instruction for safe usage..... 27

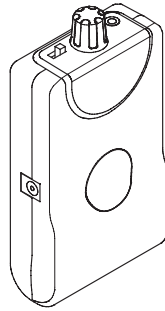
ENGLISH

1. Product Composition

1.1 Product Composition

* Make sure to check all the items are included.

Controller


Handpiece


| Handpiece is included only in one product

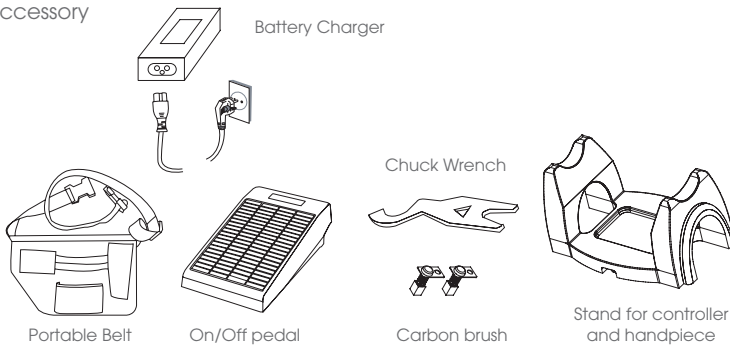Accessory


2. Examples of usage

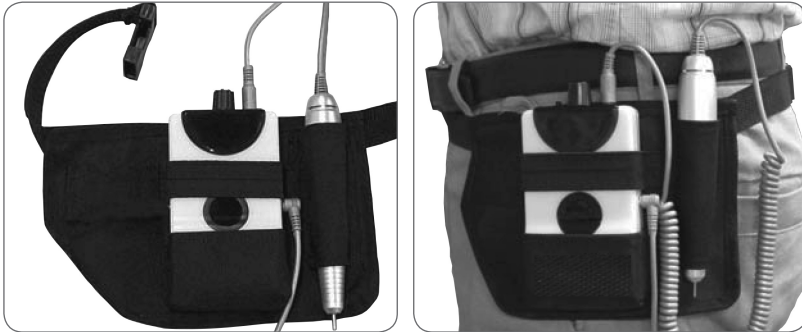2.1 For Portable


ENGLISH

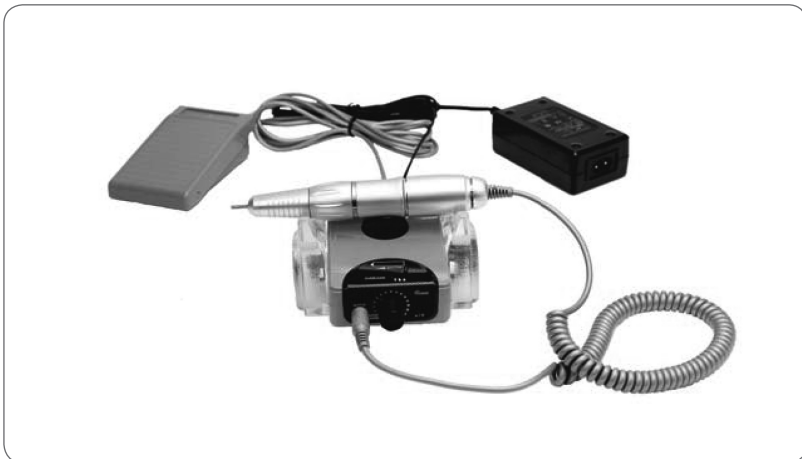2.2 For Desk


3. Operation

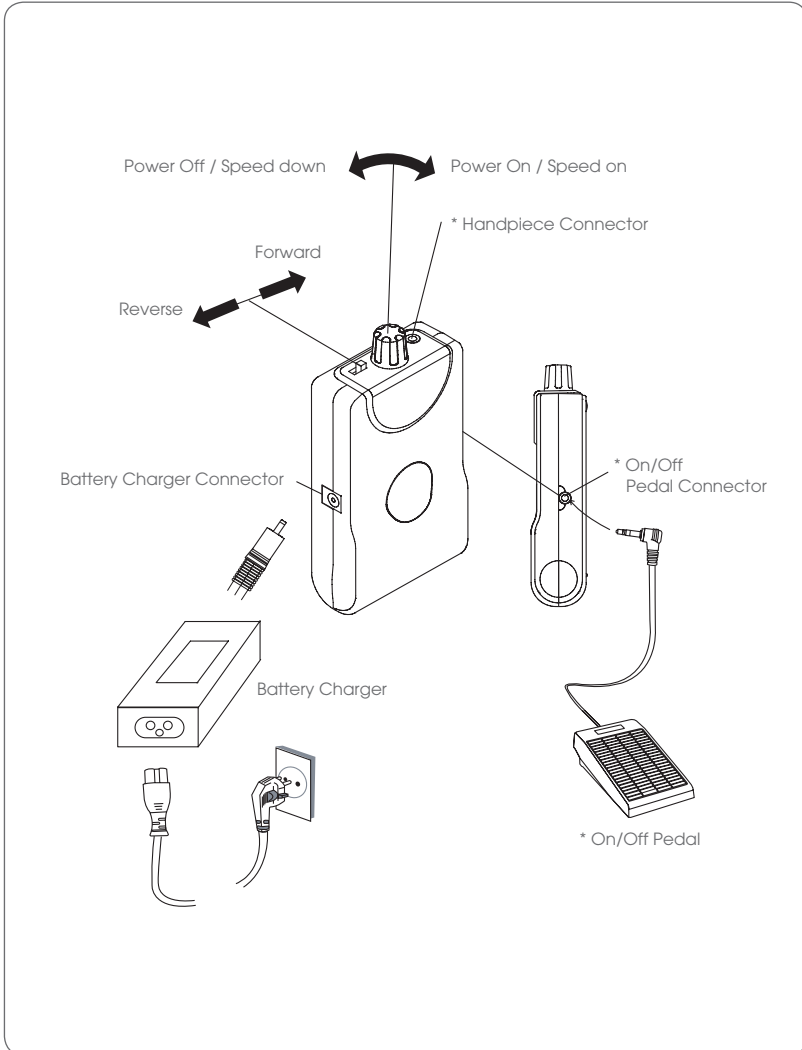3.1 Part Function

ENGLISH

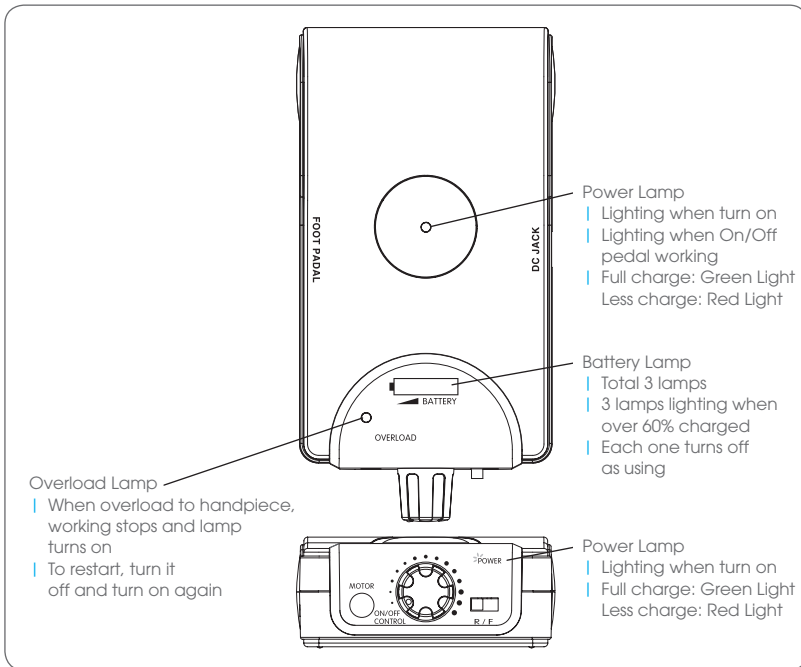3.2 LED LAMP Detail

3.3 Overload Prevention Function

Red light will turn on the overload prevention lamp when excessive power is applied on the handpiece. Turn the power down and then start again.

4. Battery

Built-in battery at the inside of controller is Lithium-Ion Polymer type battery that is rechargeable.

4.1 Battery Recharge

- ① When the red light turns on the battery lamp, please use after recharging.
- ② Connect the recharger as shown on the picture of page 7.
- ③ When recharging the battery, please keep the followings without fail.

Environmental condition when recharging

- | Temperature should be 0–40 °C
- | Place where ventilation is good
- | Place where there are no inflammables such as gasoline

Others


- | Please do not put anything on the controller
- | Please use the designated adaptor only that is included in the product
- | Please do not disassemble by yourself

4.2 Battery Exchange

Rechargeable battery has limited times of recharging therefore it needs to be exchanged after certain period of time.

Battery cannot be exchanged by the customer and must be only done by authorized service provider.

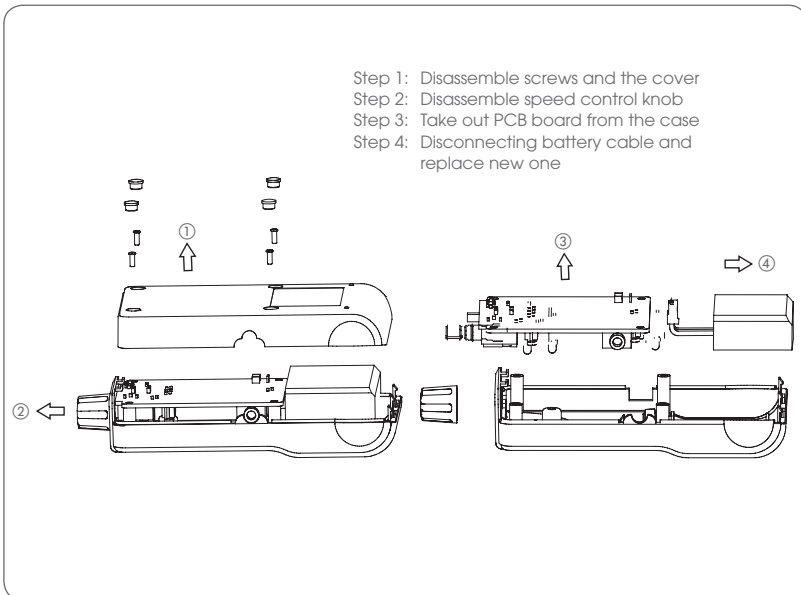For more detailed information, please contact our head office or shop where you bought.


5. Maintenance

Product maintenance manual that is shown below is for the special technician only. Customer should not fix it by own. Please contact head office or shop where you bought.

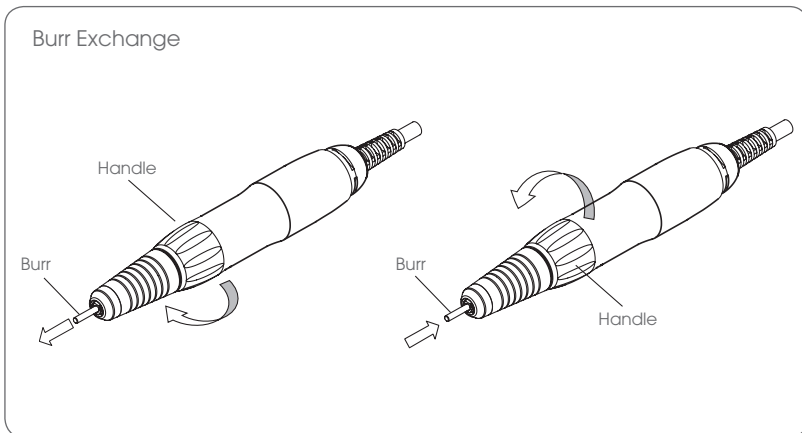
5.1 Handpiece Burr Exchange


WARNING

- | Before inserting, please check the condition of burr. Do not use the defective burr such as bent or cracked.
- | Before inserting, please make sure that the handpiece is stopped completely.
- | After inserting, please make sure that the chuck is locked. It is very dangerous to operate it when the chuck is not locked.

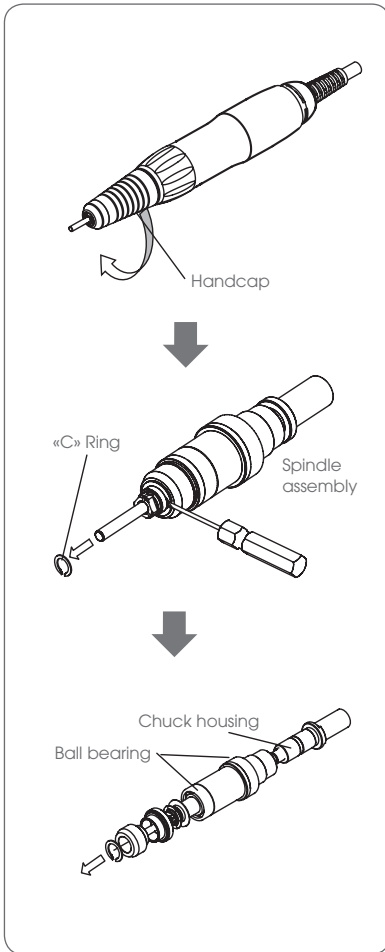
5.2 Ball Bearing Exchange

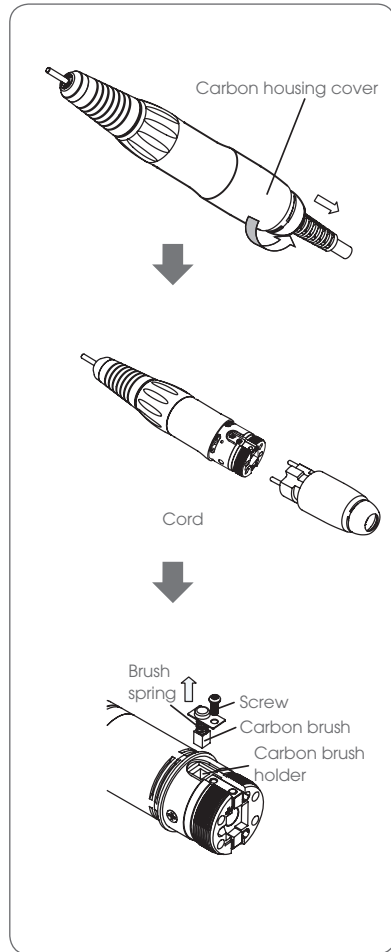
5.3 Carbon Brush Exchange

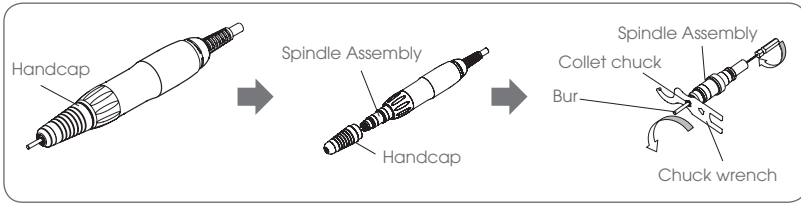5.4 Collet Chuck Exchange


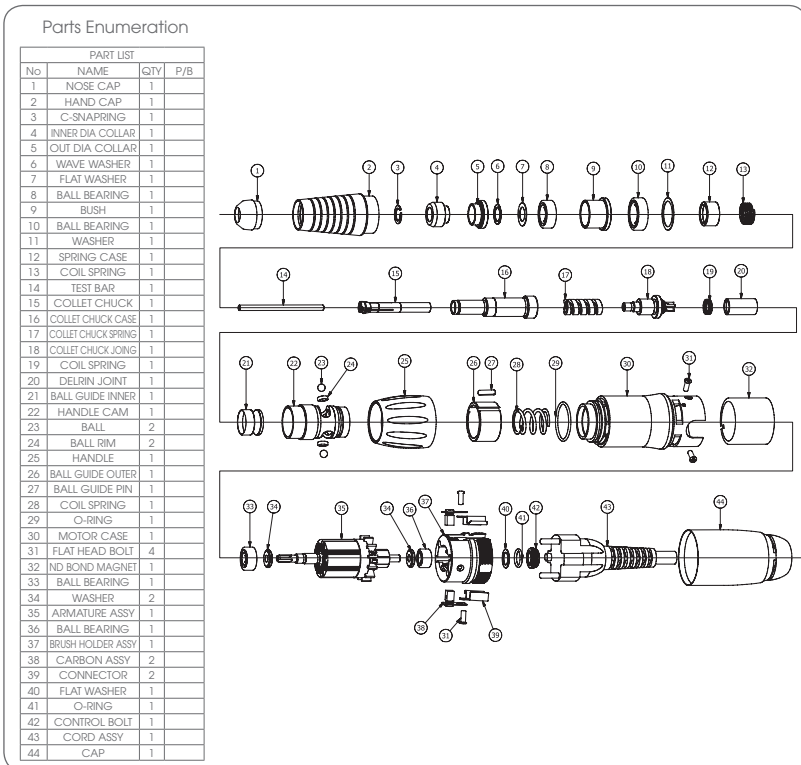
5.5 Handpiece Diagram

For the improvement of product quality, product specification and design can be changed without prior notice. However, you cannot request revision.

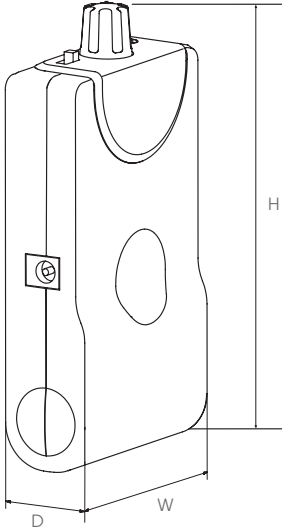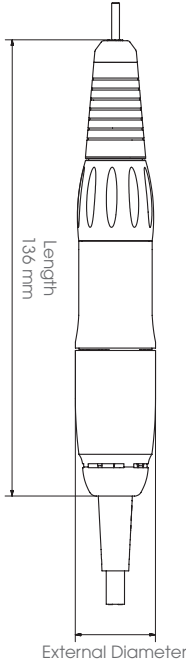6. Product Specifications

	
<ul style="list-style-type: none"> Size (WxDxH) in mm: 67x30x132 Weight: 185 g (World Best) Recharger Input Voltage AC110 – 230 V, 50/60 Hz Output Voltage: DC 13 V Charging Time: 2 h 30 m Using Time (unloaded Standard) At full speed: 5 hours At 10 000 rpm: 8.5–9 hours 	<ul style="list-style-type: none"> Size External Diameter \varnothing 24 mm, Length: 136 mm Weight: 120 g (Excluding Cable 60 g) Rotating Speed 0–24000 rpm (DC 13 V, 1.2 A) Burr Size: \varnothing 2.35 mm

ENGLISH

7. Instructions for Safety Usage

7.1 Environmental condition for using

- | Temperature: + 0 °C – + 40 °C
- | Humidity: 35%–70%
- | Pressure: under 1060 hPa
- | Place: No firearm and inflammables and good ventilation
- | Out of reach from the old and the infirm

7.2 Cautions

- | Please think customer's safety first all the time when using.
- | When you taking the product out of its package, please make sure not to drop it.
- | To avoid electric shock or related accident, please do not operate the instrument with wet hands.
- | If there is a defect on the power cable (cord), please do not use it. Exchange it with new one and then use it.
- | Please do not insert more than 2 power cable (cord) into 1 socket.
- | In order to prevent the electrical short, electrical shock and fire, please do not damage the motor cord.
- | Please check the abnormality of the product before using. If abnormal assembling condition, handpiece vibration, abnormal noise and temperature are found, then stop using it immediately and contact our head office or shop where you bought.
- | Please do not place anything on its ventilating opening.
- | Please do not blow the compressed air into ventilating opening.
- | Please do not leave this product at the situation of being operated.

- | Skin contact might cause rashes and allergic reactions especially for sensitive users. In this case, stop using it and contact dermatologist.
- | Please do not disassemble or modify it by own. It might cause the problem of its original performance and safety.
- | Please make sure not to drop the handpiece. Some parts of the handpiece are weak against the shock, therefore it might cause malfunction.
- | Please be careful not to let oil and water penetrate into the handpiece and controller.
- | Please do not use oil for cleaning the handpiece. It might become a major factor of bearing damage.
- | Please keep the handpiece in which the burr should be inserted all the time.
- | Please use the burr that is ISO standard and avoid the using of bent burr.
- | Please keep the permitted rotation speed without fail according to its burr size.
- | Please do not open the chuck when the handpiece is rotating. For exchanging the burr, please turn off its power and then wait until rotation stops completely and then exchange it.
- | When disposing the battery, please keep the international law and national laws.
- | Please do not put the battery into the fire or heat the battery due to its explosion risk.
- | When the chemical substances of battery goes into the eyes or contacts the skin, then rinse off with running water and go see a doctor.

WARRANTY

ENGLISH

PRODUCT	
Model No.	
Serial No.	
Purchase Date	
CUSTOMER	
Company	
Address	
Tel.	
E-Mail	

- | Warranty of purchased product lasts 12 month from the purchase date. (However consumption parts are excluded.)
- | Defective material and assembling failure shall be repaired immediately without any cost.
- | If any defects occur during the warranty period, you can request the repair by writing down defect content on the back side of warranty card.
- | We are not responsible for the defects caused by inappropriate treatment, installation, immoderate usage and improper repairs that were done not by the authorized personnel or distributor but by other place.
- | We are not responsible for the defects that are caused by the general abrasion.
- | We are not legally responsible for the damage caused by user's carelessness.
- | This product warranty is only effective for first purchaser (company).
- | For requesting this warranty in the future, please keep the receipts that you received when purchasing this product.
- | During the warranty period, the buyer bears the cost of returning the goods.

SIMON KELLER AG | Lyssachstrasse 83 | 3400 Burgdorf | Schweiz
T 034 420 08 00 | F 034 420 08 01 | info@simonkeller.ch | www.simonkeller.ch

SIMON KELLER Ltd. | Lyssachstrasse 83 | CH-3400 Burgdorf | Switzerland
Phone +41 34 420 08 00 | Fax +41 34 420 08 01 | info@simonkeller.ch | www.simonkeller.ch